

## Synchron® LXi 725 Access® Clinical System

Task	Resource	How Measured?	Achieved?		N/A	Date	Trainee Initial	Trainer Initial
			Y	N				
<b>Supplies</b>								
• Determine supply / reagent status for Access	LXi 725 Instructions For Use (IFU) → Chapter 3: Pre-Test Activities	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Determine supply / reagent status for CTA		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Determine supply / reagent status for LX20		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Load supplies/reagents on Access		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Load supplies/reagents on CTA		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Load supplies/reagents on LX20		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
<b>Calibration</b>								
• Determine calibration status for Access immunoassays	LXi 725 Instructions For Use (IFU) → Chapter 4: System Calibration	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Determine calibration status for DxC chemistries		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Program and process calibration on Access immunoassays		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Program and process calibration for LX20 chemistries		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			

## Synchron® LXi 725 Access® Clinical System

Task	Resource	How Measured?	Achieved?		N/A	Date	Trainee Initial	Trainer Initial
			Y	N				
<b>Sample Processing</b>								
• Identify sample requirements	Access Assay Manual or Synchron Chemistry Information Manual	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Determine correct sample rack/ container for sample processing at LXi	LXi 725 Instructions For Use (IFU) → Chapter 3: Pre-Test Activities, Preparing Test Samples	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Program and process routine and STAT patient samples at LXi	LXi 725 Instructions For Use (IFU) → Chapter 5: Sample Programming and Processing	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Locate completed test results	LXi 725 Instructions for Use (IFU), Chapter 2: System Description, Recalling Test Results	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
<b>Quality Control</b>								
• Program and process QC at DxC 600i	LXi 725 Instructions for Use (IFU) → Chapter 4: System Calibration (Running QC Samples)	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Define Quality Control	LX20 Operations Manual Chapter 10: Quality Control → Defining a Control	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• QC Results Management	LX20 Operations Manual Chapter 10: Quality Control	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
<b>Maintenance</b>								
• Perform daily maintenance on Access	From the Sample Manager Screen ↳ Help	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			

## Synchron® LXi 725 Access® Clinical System

Task	Resource	How Measured?	Achieved?		N/A	Date	Trainee Initial	Trainer Initial
			Y	N				
	↳Requesting Routine Maintenance ↳Daily Maintenance							
• Perform twice weekly maintenance on LX20	LX20 Maintenance Manual: Chapter 2: Twice Weekly Maintenance	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Perform weekly maintenance on Access	From the Sample Manager Screen ↳Help ↳Requesting Routine Maintenance ↳Weekly Maintenance	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Perform weekly maintenance on LX20	LX20 Maintenance Manual: Chapter 2: Weekly Maintenance	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Perform Twice-monthly maintenance on CTA	LX20 Maintenance Manual: Chapter 2: Twice Monthly Maintenance	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Perform Monthly Maintenance on DxC	LX20 Maintenance Manual: Chapter 2: Monthly Maintenance	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Perform 2-Month Maintenance on DxC	LX20 Maintenance Manual: Chapter 2: Maintenance	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Perform 3-Month Maintenance on DxC		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Perform 4-Month Maintenance on DxC		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
• Perform 6-Month Maintenance on DxC		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			

## Synchron® LXi 725 Access® Clinical System

Task	Resource	How Measured?	Achieved?		N/A	Date	Trainee Initial	Trainer Initial
			Y	N				
<ul style="list-style-type: none"> <li>Perform Yearly Maintenance on CTA</li> </ul>		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
<ul style="list-style-type: none"> <li>Perform As-Needed Maintenance on CTA</li> </ul>								
<ul style="list-style-type: none"> <li> <ul style="list-style-type: none"> <li>Flush CTA Aliquot Probe</li> </ul> </li> </ul>	LXi 725 Instructions for Use (IFU) → Chapter 7: Maintenance → CTA Maintenance	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
<ul style="list-style-type: none"> <li> <ul style="list-style-type: none"> <li>Replacing the CTA Pick and Place Head</li> </ul> </li> </ul>		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
<b>Miscellaneous</b>		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
Instrument Setup at LXi 725	LXi 725 Instructions for Use → Chapter 5: LXi Setup Options	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
Quality Control Setup/Definition at LXi 725	LX20 Operations Manual → Chapter 10: Quality Control	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
Quality Control Results Management at LXi 725	LX20 Operations Manual → Chapter 10: Quality Control	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
Operate system in Independent Mode (LX20, CTA or Access non-operational)	LXi 725 Instructions for Use (IFU) → Appendix A → Independent Operating Modes	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
Power down/power up Access, CTA and DxC	DxC 600i Instructions for Use (IFU) → Chapter 6 → Post Test Activities	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
Initialize Access	DxC 600i Instructions for Use (IFU) → Chapter 6 → Post Test Activities	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
Home CTA and DxC	DxC 600i Instructions for Use (IFU) → Chapter 6 → Post Test Activities	<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			

## Synchron® LXi 725 Access® Clinical System

Task	Resource	How Measured?	Achieved?		N/A	Date	Trainee Initial	Trainer Initial
			Y	N				
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			
		<input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>			