




free T3

- Improved result harmonization to commercially available free T3 assays
- Minimized response to heterophile antibodies
- Sample type flexibility (serum or heparin plasma)
- Time to first result: approximately 25 minutes
- Ready-to-use liquid calibrators, stable at 2-10°C for 4 months
- Open-pack and calibration stability: 28 days

2 step
labeled antibody
technique

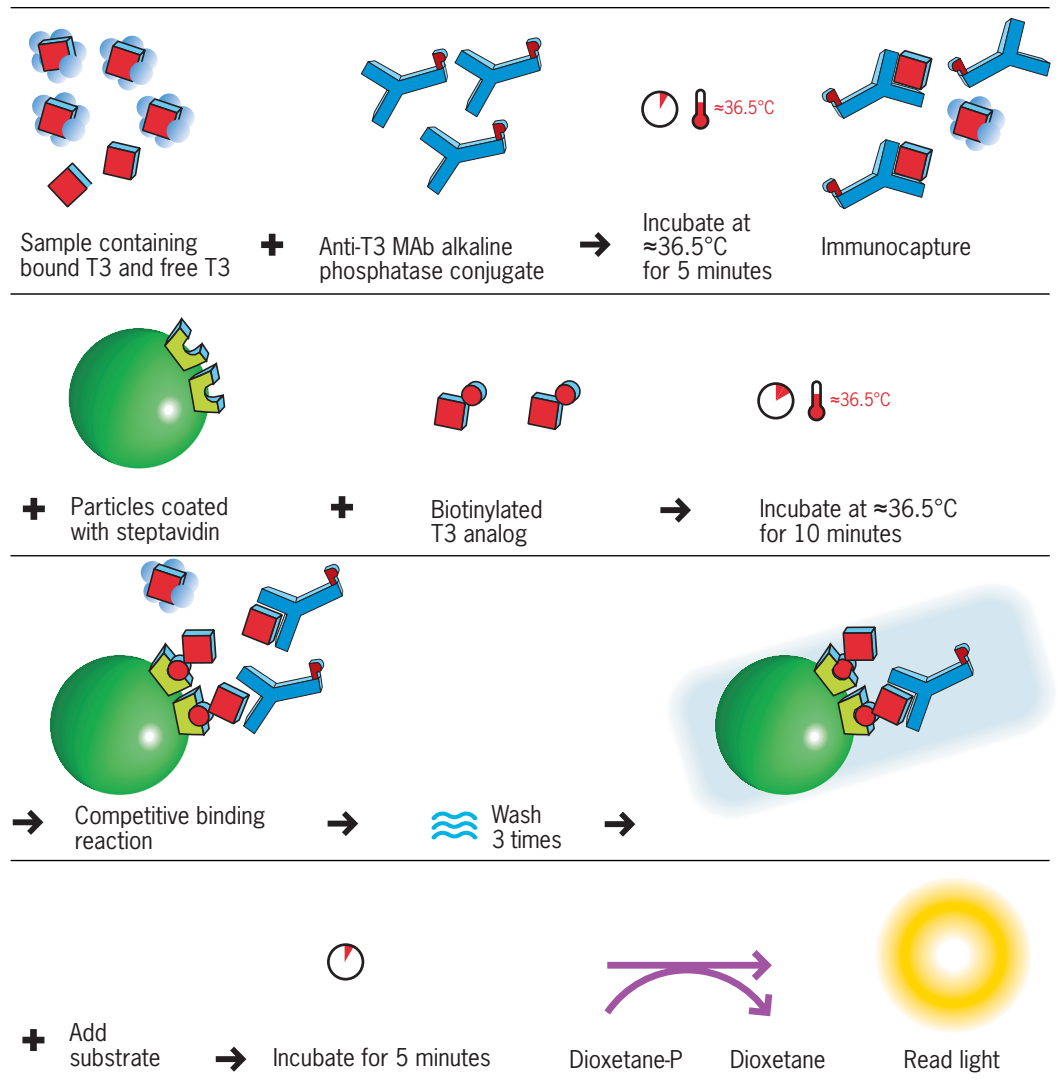

Signal produced is inversely proportional to the free T3 concentration in the sample.






free T3

Performance Correlation with predicate device

A comparison of 397 values using the Access® free T3 assay on the Access Immunoassay system and a commercially available immunoassay kit gave the following statistical data, using Deming calculations.

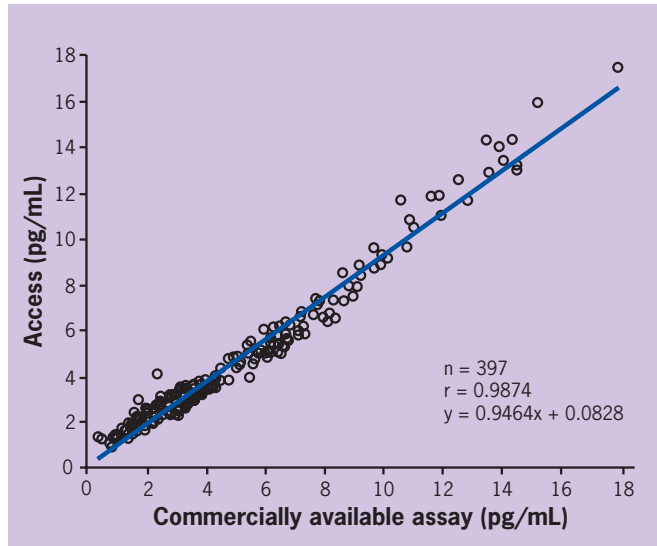

Characteristics

Sample Type/Size	Serum or plasma (heparin)/55 µL
Time to First Result	25 minutes
Analytical Sensitivity	0.88 pg/mL (1.4 pmol/L)
Calibrator Levels	0, approximately 1, 2, 5, 10 and 30 pg/mL (0, 1.5, 3.1, 7.7, 15 and 46 pmol/L)
Expected Range	2.5 – 3.9 pg/mL (3.8 – 6.0 pmol/L)
Open Pack Stability	28 days
Calibration Stability	28 days
Precision	≤ 12% at concentrations greater than 2.0 pg/mL

Ordering Information

Access® free T3 - 2 packs of 50 tests/pack	A13422
Access® free T3 Calibrators - 6 vials of 2.5 mL/vial	A13430



Eastern Europe, Middle East, North Africa, South West Asia: Switzerland, Nyon (41) 22 994 0707.
 Australia, Gladesville (61) 2 9844 6000. Canada, Mississauga (1) 905 819 1234. China, Beijing (86) 10 6515 6028.
 Hong Kong (852) 2814 7431, 2814 0481. France, Villepinte (33) 1 49 90 90 00. Germany, Krefeld (49) 2151 33 35.
 Italy, Cassina de' Pecchi (Milan) (39) 02 953921. Japan, Tokyo (81) 3 5404 8424. Latin America (1) (305) 380 4709.
 Mexico, Mexico City (52) 55 560 57770. Netherlands, Mijdrecht (31) 297 230630. Puerto Rico (787) 747 3335.
 Singapore (65) 6339 3633. South Africa/Sub-Saharan Africa, Johannesburg (27) 11 805 2014. Sweden, Bromma (46) 8 564 85 900.
 Switzerland, Nyon 0800 850 810. Taiwan, Taipei (886) 2 2378 3456. Turkey, Istanbul (90) 216 309 1900.
 UK, High Wycombe (44) 01494 441181. USA, Brea, CA (1) 800 352 3433, (1) 714 993 5321.