


Rubella IgM

- *IgM immunocapture assay*
 - *Early detection of seroconversions*
 - *No competition with specific IgG*
 - *No interference with rheumatoid factor*
- *Ready-to-use reagents, including the conjugate*

2 step IgM immunocapture technique

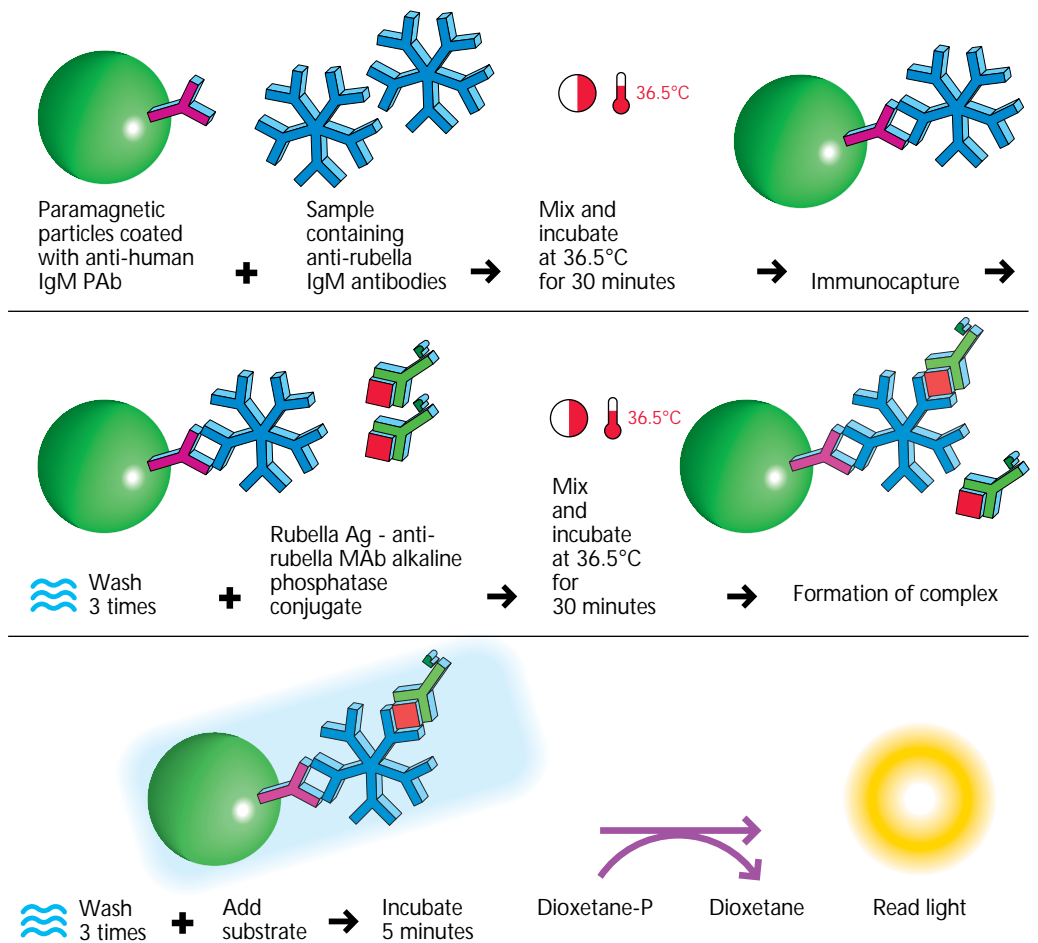

Signal produced is directly proportional to the anti-Rubella IgM virus concentration in the sample



Performance

Rubella IgM

Access® Rubella IgM - reference EIA technique concordance

Access®	ELISA technique	
	Negative	Positive
Nonreactive 0 - < 10 AU/mL	243	2*
Equivocal ≥ 10 - < 15 AU/mL	0	1*
Reactive ≥ 15 AU/mL	0	27*

**Access® : 100% specificity, 100% sensitivity
(versus clinical status)**

* These three sera were collected more than 60 days after rash; Access® results express the fast IgM decrease and the non-persistence of residual IgM.

Characteristics

Sample Type/Size	Serum/20 µL
Time to 1st Result	75 minutes
Control Levels	0, 5, 15, and 60 AU/mL
Interpretation	0 - <10 AU/mL non-reactive ≥ 10 - < 15 AU/mL equivocal ≥ 15 AU/mL reactive
Open Pack Stability	28 days
Calibration Stability	28 days

Ordering Information

Access® Rubella IgM - 2 packs of 50 tests/pack	34440
Access® Rubella IgM Controls - 4 vials of 1 mL/vial	34445
Access® QC Rubella IgM - 2 levels, 3 vials/level of 2.5 mL/vial	34449



Africa/Middle East/Eastern Europe: Switzerland, Nyon (41) 22 994 0707. Australia, Gladesville (61) 2 9844 6000. Canada, Mississauga (1) 905 819 1234. China, Beijing (86) 10 6515 6028. Hong Kong (852) 2814 7431, 2814 0481. France, Villepinte (33) 1 49 90 90 00. Germany, Krefeld (49) 2151 33 35. Italy, Milan (39) 02 953921. Japan, Tokyo (81) 3 5404 8424. Latin America (305) 380 4709. Mexico, Mexico City (525) 575 6805. Netherlands, Mijdrecht (31) 297 230630. Singapore (65) 339 3633. South Africa, Johannesburg (27) 11 805 2014. Spain, Madrid (34) 91 728 7900. Sweden, Bromma (46) 8 98 53 20. Switzerland, Nyon 0800 850 810. Taiwan, Taipei (886) 2 2378 3456. UK, High Wycombe (44) 01494 441181. USA, Brea, CA 800 352 3433 714 993 5321.