



IMMAGE Immunochemistry Systems Instructions For Use

© 2025 Beckman Coulter, Inc. All rights reserved.

WASH
Wash Solution

REF 447060

For *In Vitro* Diagnostic Use

FOR PROFESSIONAL USE ONLY

Rx Only

PRINCIPLE

INTENDED USE

Wash Solution is intended for use with IMMAGE Immunochemistry Systems and reagents for the quantitative determinations of components in biological fluids.

REAGENTS

CONTENTS

Each kit contains the following items:

KIT COMPONENTS	QUANTITY
IMMAGE Immunochemistry Systems Wash Solution	1

REACTIVE INGREDIENTS

REAGENT CONSTITUENTS	VOLUME
IMMAGE Immunochemistry Systems Wash Solution (Capso-buffered Saline)	10 L

GHS HAZARD CLASSIFICATION

Not classified as hazardous


	Safety Data Sheet is available at beckmancoulter.com/techdocs
--	---

MATERIALS NEEDED BUT NOT SUPPLIED WITH REAGENT KIT

IMMAGE Immunochemistry Systems

STORAGE AND STABILITY

WASH is stable until the expiration date on the label if stored at room temperature with the screw cap in place. Room temperature storage is represented by this symbol on the label.

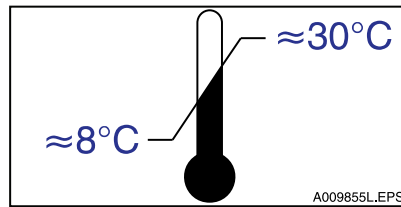

Wash Solution storage location:

TESTING PROCEDURE(S)

After setup, load reagents onto the system as directed in the *IMMAGE Operations Manual*.

ADDITIONAL INFORMATION

For more information, refer to the *IMMAGE Immunochemistry Systems Operations Manual*.

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

May be covered by one or more pat. -see www.beckmancoulter.com/patents.

For a patient/user/third party in the European Union and in countries with identical regulatory regime (*Regulation 2017/746/EU on In vitro Diagnostic Medical Devices*); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

SHIPPING DAMAGE

If damaged product is received, notify your Beckman Coulter Clinical Support Center.

REVISION HISTORY

Revision AE

Added new language requirement: Norwegian.

Revision AF

Updated corporate address. Added sodium azide caution statement (in Reagents).

Revision AG

Added Revision History

Revision AH

Added new language requirement: Czech, and Korean.

Revision AJ

Added GHS classification and Trademark statement.

Revision AK

Added new language requirement: Romanian

Revision AL

Updates to comply with requirements per Beckman Coulter Global Labeling Policy.

Revision AM

Added new language requirement: Brazilian Portuguese.

Revision AN

Added new language requirement: Dutch, Romanian, and Slovak. Additional changes to comply with requirements per Beckman Coulter Global Labeling Policy.

Revision AP

Updates to comply with IVDR requirements per Beckman Coulter Global Labeling Policy.













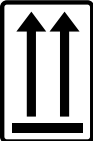




Revision AR

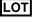Updates to comply with requirements per Beckman Coulter Global Labeling Policy.


Added new language requirement: Bulgarian, Croatian, Estonian, Finnish, Indonesian, Latvian, Russian, Slovenian, and Ukrainian.

SYMBOLS KEY

Table 1.0

	Catalogue Number		In Vitro Diagnostic
	Contents		Temperature limit
	Manufacturer		Expiration Date
	Batch code		Safety Data Sheet
	CE Mark		Consult Instructions for Use
	Authorized Representative in the European Community		Date of Manufacture
	This way up		Do not stack more than 4 high
	Made in Germany		Do not stack more than 5 high
	Made in USA of US and Foreign Components		Made in USA of US and Foreign Components

 Beckman Coulter Ireland Inc., Lismeehan, O'Callaghan's Mills, Co. Clare, Ireland +(353) (0) 65 683 1100

 Beckman Coulter, Inc., 250 S. Kraemer Blvd., Brea, CA 92821 U.S.A.
 +(1) 800-854-3633
www.beckmancoulter.com