COULTER CLENZ Cleaning Agent

REF 8546931, 8546929, 8546930

TABLE OF CONTENTS

English	2
Español	4
Português (Portugal)	6
한국어	8
Português (Brasil)	10
Tiếng Việt	12
REFERENCES	14

COULTER CLENZ Cleaning Agent

REF 8546931, 8546929, 8546930

**For *In Vitro* Diagnostic Use
Rx Only in the U.S.A.**

INTENDED USE

For use as a cleaning agent for COULTER hematology analyzer components that come in contact with blood samples, including apertures and blood sampling valves. Refer to your instrument product manuals and/or online help, as applicable.

Not intended for use on the COULTER LH 780 hematology analyzer.

SUMMARY

This biodegradable and azide-free reagent contains a proteolytic enzyme that aids in the removal of protein buildup in the sensing orifices of a hematology analyzer.

PRINCIPLE

Blood proteins flowing through blood counting orifices (apertures) in an instrument over time accumulate around these openings to a point that the size of the orifices are minimized and eventually blocked. The proteolytic enzyme in the cleaning agent reacts with and dissolves the protein in and around the apertures.¹

REACTIVE INGREDIENTS

A solution of a proteolytic enzyme

WARNING AND PRECAUTIONS

- Do not inhale and/or ingest.
- Avoid eye and skin contact. In case of eye or skin contact, flush affected area with copious amounts of water for at least 15 minutes.
- DO NOT REUSE CONTAINERS.

GHS HAZARD CLASSIFICATION		
COULTER CLENZ	DANGER	Avoid breathing vapours.
		May cause allergy or asthma symptoms or breathing difficulties if inhaled.
		May cause an allergic skin reaction.
		Wear protective gloves, protective clothing and eye/face protection.
		In case of inadequate ventilation, wear respiratory protection.
		IF INHALED: Remove person to fresh air and keep at rest in a position comfortable for breathing.
		If experiencing respiratory symptoms: Call a POISON CENTER or doctor/physician.
		Diazolidinyl Urea <1%
		Subtilisin <1%
		Reaction Mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC# 247-500-7] and 2-methyl-4-isothiazolin-3-one [EC# 220-239-6](3:1) <0.0015%

STORAGE, STABILITY, AND DISPOSAL

- Store COULTER CLENZ Cleaning Agent at 2-25°C.
- Do not use product past expiration date.
- Use product at temperatures stated in the instrument product manuals and/or online help.
- Opened containers are stable for 3 months when stored as recommended.
- Dispose of waste product, unused product, and contaminated packaging in compliance with federal, state, and local regulations.

PREPARATION

COULTER CLENZ Cleaning Agent is ready to use. Replace the reagent container as directed in your instrument product manuals and/or online help.

IMPORTANT: If product has been partially or completely frozen, allow product to warm to room temperature. Mix product by gentle inversion prior to placement on the instrument. Install the reagent, and prime if necessary, as directed in your instrument product manuals and/or online help.

PRODUCT AVAILABILITY

COULTER CLENZ Cleaning Agent - 1 x 500 mL

REF

8546929

COULTER CLENZ Cleaning Agent - 1 x 5 L

REF

8546930

COULTER CLENZ Cleaning Agent - 1 x 10 L

REF

8546931

TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

For additional information or if damaged product is received, call Beckman Coulter Customer Service at 800-526-7694 (USA or Canada) or contact your local Beckman Coulter Representative.

ADDITIONAL INFORMATION

Glossary of Symbols is available at techdocs.beckmancoulter.com (PN C07163)

SDS

Safety Data Sheet is available at techdocs.beckmancoulter.com

Revision History

Revision AD, 10/2014

Changes were made to:

- GHS HAZARD CLASSIFICATION

Revision AE, 02/2017

Changes were made to:

- TRADEMARKS
- ADDITIONAL INFORMATION
- Added new Brazilian Portuguese language
- Added new Brazilian address

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Beckman Coulter, Inc., 250 S. Kraemer Blvd., Brea, CA 92821 U.S.A.