# COULTER CLENZ Cleaning Agent

**REF** 8448222, 8448188

## TABLE OF CONTENTS

English . . . . .	2
Français (FR) . . . . .	4
Deutsch (DE) . . . . .	6
Italiano (IT) . . . . .	8
Español (ES) . . . . .	10
Português Portugal (PT-PT) . . . . .	12
Dansk (DA) . . . . .	14
Svenska (SV) . . . . .	16
Norsk (NO) . . . . .	18
Suomi . . . . .	20
Ελληνικά (EL) . . . . .	22
Lietuviškai (LT) . . . . .	24
Magyar (HU) . . . . .	26
Polski (PL) . . . . .	28
Čeština (CZ) . . . . .	30
Türkçe (TR) . . . . .	32
Русский (RU) . . . . .	34
Hrvatski (HR) . . . . .	36
Română (RO) . . . . .	38
Србија (SR) . . . . .	40
Українська (UK) . . . . .	42
Português Brasil (PT-BR) . . . . .	44
Nederlands . . . . .	46
REFERENCES . . . . .	48

---

# COULTER CLENZ Cleaning Agent

**REF** 8448222, 8448188

**For *In Vitro* Diagnostic Use  
Rx Only in the U.S.A.**

## INTENDED USE

For use as a cleaning agent for COULTER hematology analyzer components that come in contact with blood samples, including apertures and blood sampling valves. Refer to your instrument product manuals and/or online help, as applicable.

Not intended for use on the COULTER LH 780 hematology analyzer.

## SUMMARY

This biodegradable and azide-free reagent contains a proteolytic enzyme that aids in the removal of protein buildup in the sensing orifices of a hematology analyzer.

## PRINCIPLE

Blood proteins flowing through blood counting orifices (apertures) in an instrument over time accumulate around these openings to a point that the size of the orifices are minimized and eventually blocked. The proteolytic enzyme in the cleaning agent reacts with and dissolves the protein in and around the apertures.<sup>1</sup>

## REACTIVE INGREDIENTS

A solution of proteolytic enzyme

## WARNING AND PRECAUTIONS

- Do not inhale and/or ingest.
- Avoid eye and skin contact. In case of eye or skin contact, flush affected area with copious amounts of water for at least 15 minutes.
- DO NOT REUSE CONTAINERS.

GHS HAZARD CLASSIFICATION	
COULTER CLENZ Cleaning Agent	DANGER
	
	May cause an allergic skin reaction.
	May cause allergy or asthma symptoms or breathing difficulties if inhaled.
	Avoid breathing vapours.
	Wear protective gloves, protective clothing and eye/face protection.
	In case of inadequate ventilation, wear respiratory protection.
	IF INHALED: Remove person to fresh air and keep at rest in a position comfortable for breathing.
	If experiencing respiratory symptoms: Call a POISON CENTER or doctor/physician.
	Diazolidinyl Urea <1% Subtilisin <1% Reaction Mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC# 247-500-7] and 2-methyl-4-isothiazolin-3-one [EC# 220-239-6](3:1) <0.0015%

## STORAGE, STABILITY, AND DISPOSAL

- Store COULTER CLENZ Cleaning Agent at 2-25°C.
- Do not use product past expiration date.
- Use product at temperatures stated in the instrument product manuals and/or online help.
- Opened containers are stable for 3 months when stored as recommended.
- Dispose of waste product, unused product, and contaminated packaging in compliance with federal, state, and local regulations.

## PREPARATION

COULTER CLENZ Cleaning Agent is ready to use. Replace the reagent container as directed in your instrument product manuals and/or online help.

**IMPORTANT:** If product has been partially or completely frozen, allow product to warm to room temperature. Mix product by gentle inversion prior to placement on the instrument. Install the reagent, and prime if necessary, as directed in your instrument product manuals and/or online help.

## PRODUCT AVAILABILITY

COULTER CLENZ Cleaning Agent - 1 x 1 L

**REF** 8448188

COULTER CLENZ Cleaning Agent - 1 x 5 L

**REF** 8448222

## TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

For additional information or if damaged product is received, call Beckman Coulter Customer Service at 800-526-7694 (USA or Canada) or contact your local Beckman Coulter Representative.

## ADDITIONAL INFORMATION

Glossary of Symbols is available at [techdocs.beckmancoulter.com](http://techdocs.beckmancoulter.com) (PN C07163)

May be covered by one or more pat. - see [www.beckmancoulter.com/patents](http://www.beckmancoulter.com/patents)

**SDS** Safety Data Sheet is available at [techdocs.beckmancoulter.com](http://techdocs.beckmancoulter.com)

## Revision History

Revision AF, 01/2018

Changes were made to:

- Added new Serbian language

Revision AG, 04/2018

Changes were made to:

- Added new Ukrainian language

Revision AH, 09/2018

Changes were made to:

- Added new Patent Information Statement
  - Added new Dutch, Finnish, Croatian and Serbian languages
-

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Revisie AG, 04-2018
Beckman Coulter, Inc., 250 S. Kraemer Blvd., Brea, CA 92821 U.S.A.