



CUSTOMER COMMUNICATION

All AU/DxC AU Clinical Chemistry Analyzers
Apolipoprotein (Apo) B, REF OSR6143, Lot 2629 and Lot 2630
and Apo A1 & B Calibrator, REF ODR3022, Lots 280X

April 2024

Dear Beckman Coulter AU and DxC AU Customer,

Beckman Coulter would like to inform you that Apo B reagent (OSR6143), Lot 2629 and Lot 2630 are only value assigned to Apo A1 & B Calibrator (ODR3022) Lots 2900 and higher. ODR3022 Lots 2802, 2803, 2804, 2805 and 2806 must not be used with OSR6143, Lot 2629 and Lot 2630.

Please use Apo A1 & B Calibrator **ODR3022, Lot 2900 and higher** to calibrate Apo B reagent **OSR6143, Lot 2629 and 2630**.

Any other alternate/in-date Apo B reagent OSR6143 (e.g. Lots 2626, 2627, 2628) can be calibrated with ODR3022, Lots 2802, 2803, 2804, 2805 and 2806.

Please share this information with your laboratory staff and retain this notification as part of your laboratory documentation.

If you have any questions regarding this product, please contact your local Beckman Coulter Representative, or use the following link for a listing of local contact information.

<https://www.beckmancoulter.com/en/support/contact-us>

Thank you for your continued support of Beckman Coulter products and services.

Sincerely,

A handwritten signature in black ink that reads 'Noretta Clifford'.

Noretta Clifford
Product Manager

@2024 Beckman Coulter Inc. All rights reserved. Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries. All other trademarks are the property of their respective owners.

For Beckman Coulter's worldwide office locations and phone numbers, please visit www.beckmancoulter.com/contact